



# **Modvel Knee Brace**

COMPRESSION SLEEVES

---

**USER MANUAL**

Welcome to the Knee Brace Compression Sleeve User Manual. Our Knee Brace Compression Sleeve is designed to provide medical-grade compression to help improve circulation, relieve pain, and support your knees during physical activity. Whether you're a professional athlete or just looking to stay active, our Knee Brace Compression Sleeve will help you perform at your best.

## TABLE OF CONTENTS

• <b>Product Features</b>	1
• <b>Knee Brace Benefits</b>	2
• <b>Design &amp; Comfort</b>	3
• <b>Usage Guide</b>	4
• <b>Recommended Uses</b>	5
• <b>Care Instructions</b>	6
• <b>Safety Information</b>	7
• <b>More Products</b>	8
• <b>New Release</b>	9

## PRODUCT FEATURES



### Medical-grade compression (Compresión de grado médico)

Designed to provide optimal compression to improve circulation and support the knee joint. This helps to reduce inflammation and relieve pain.



### Improved circulation (Mejora de la circulación sanguínea)

The compression helps to increase blood flow to the knee, bringing oxygen and nutrients to the area to help speed up injury recovery.



### Pain relief (Alivio inmediato del dolor)

The compression helps to relieve knee pain, whether it's due to injury, arthritis, or other conditions.



## WHY USE KNEE BRACE?



### Improved athletic performance

(Mejora del rendimiento atlético)

The compression and support provided by the Knee Brace Compression Sleeve can help improve your athletic performance by reducing pain and inflammation.



### Faster injury recovery

(Recuperación más rápida de las lesiones)

By improving circulation and reducing pain, the Knee Brace Compression Sleeve can help speed up injury recovery.



### Reduced knee pain for daily activities

(Reducción del dolor en la rodilla en actividades diarias)

Whether you're going for a run or just doing household chores, the Knee Brace Compression Sleeve can help relieve knee pain, allowing you to stay active and pain-free.

## DESIGN & COMFORT

The Knee Brace Compression Sleeve features a sleek and ergonomic design, making it comfortable to wear for extended periods of time.



### Anti-slip silicone strip

(Compresión de grado médico)

ensures that the sleeve stays in place during physical activity



### Moisture-Wicking Fabric

(Compresión de grado médico)

keeps you cool and dry all day and night.



## EASY TO USE

### STEP 1



Pull the brace over your calf and fold it to half

Paso 5: Desliza la manga sobre tu pantorrilla y plégala a la mitad.

### STEP 2



After folding the brace to half, then pull it up

Paso 6: Después de plicar la manga a la mitad, deslízala hacia arriba.

### STEP 3



Put it over your knee cap

Paso 7: Colócala sobre la rodilla.


### STEP 4



Fold the brace over at the top and from the bottom

Paso 8: Pliega la manga en la parte superior y en la parte inferior.

### STEP 5

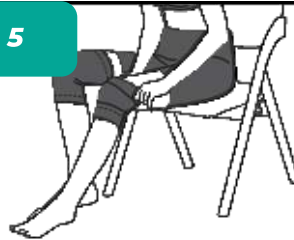

Put the folded brace up over your knee cap

Paso 9: Coloca la manga plegada sobre la rodilla.

### STEP 6



Unfold your knee brace and enjoy!

Paso 10: Despliega la manga para la rodilla y ¡disfrútalo!



## MORE COLORS AVAILABLE FOR YOUR DAILY STYLE!



## RECOMMENDED USES

The Knee Brace Compression Sleeve is recommended for a variety of activities, including running, cycling, basketball, and other high-impact sports. It is also great for daily activities such as household chores and gardening.




## SAFETY INFORMATION



The Knee Brace Compression Sleeve is made with Nylon, Spandex, and Latex material and has been FDA-approved for safety and effectiveness. If you experience any discomfort while wearing the sleeve, please remove it immediately and consult with a medical professional.

Note: This user manual should be kept with the product for reference. If you have any questions or concerns, please contact us directly at [support@modvel.com](mailto:support@modvel.com)

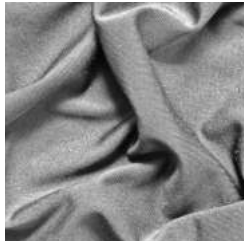

La manga de compresión para rodilla está hecha de materiales Nylon, Spandex y Latex y ha sido aprobada por la FDA para la seguridad y eficacia. Si experimenta cualquier incomodidad mientras usa la manga, retírela inmediatamente y consulte a un profesional médico.



Nota: Este manual del usuario debe guardarse con el producto como referencia. Si tiene alguna pregunta o inquietud, contáctenos directamente en [support@modvel.com](mailto:support@modvel.com).





## CARE INSTRUCTIONS

To keep your Knee Brace Compression Sleeve in top condition, machine wash in cold water and hang to dry. Avoid using fabric softeners or bleach, as these can damage the material and affect the compression properties of the sleeve.

Para mantener tu manga de compresión para rodilla en óptimas condiciones, lávala en agua fría en la lavadora y cuelga para secar. Evita usar suavizantes de telas o cloro, ya que estos pueden dañar el material y afectar las propiedades de compresión de la manga.

# ARE YOU READY FOR MORE?



## ANKLE BRACE COMPRESSION SLEEVES

- ✓ Ankle Pain Relief
- ✓ Plantar Fasciitis
- ✓ Full Ankle Support
- ✓ Comfort All Day



## Gel Knee Sleeve - Compression Support

- ✓ Knee Pain Relief
- ✓ Maximum Support
- ✓ Fast Recovery
- ✓ Designed for All Day Comfort



## Elbow Brace - Compression Support

- ✓ For Elbow Pain & Injuries
- ✓ Supportive, non-restrictive
- ✓ Lightweight & Breathable
- ✓ All-Day Use



## Knee Brace w/ Side Stabilizers & Patella Gel Pads

- ✓ For Knee Joint Pain & Injuries
- ✓ Maximum Knee Comfort
- ✓ Gel Pads for Optimal Support
- ✓ Knee Joint Protection



## Gel Enhanced Seat Cushion with True High Density Memory Foam.

- ✓ Promotes healthy posture
- ✓ Long-lasting comfort
- ✓ Lightweight & Breathable
- ✓ All-Day Use



## Back Brace for Lower Back Pain and back support.

- ✓ Back Pain Relief
- ✓ Extra Lumbar Pad for support
- ✓ Quality Fabric
- ✓ Designed for All Day Comfort