




MODVEL 2 Pack Knee Brace

Knee Compression Sleeve for Men & Women

Easy User Guide


Step Details:

Step 1

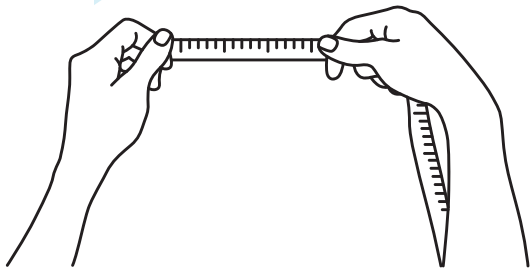


Use a Measuring Tape to measure your thigh circumference

Step 2

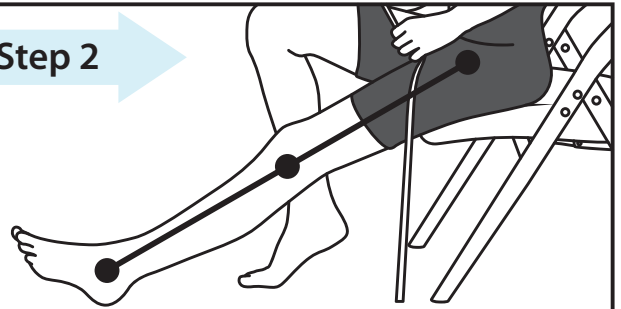


Sit on the edge of the chair with your leg straight and heel on the floor

Step 3

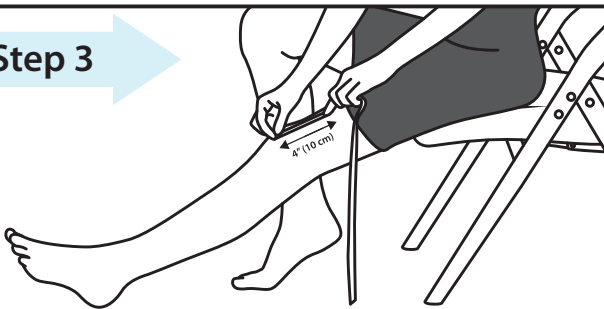


Place the end of the measuring tape on your knee cap and extend it up 4" (10 cm)

Step 4

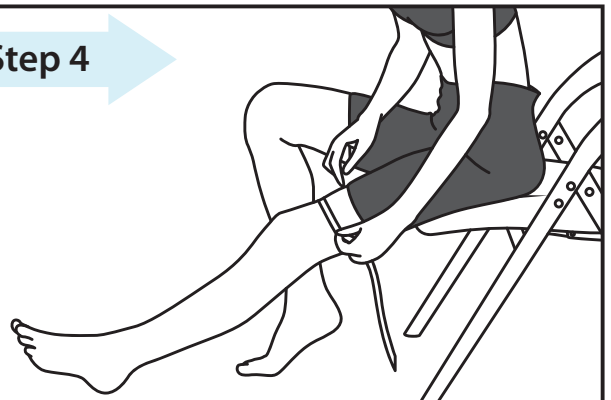


Find the circumference around your thigh

Step 5




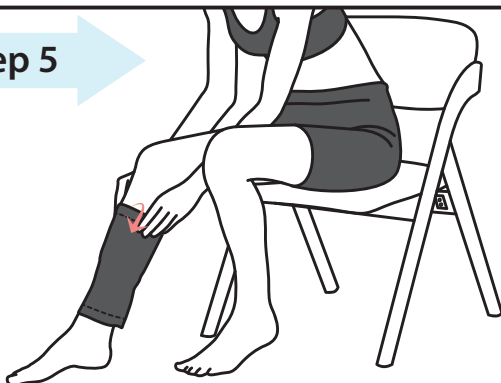


Pull the brace over your calf and fold it to half

Step 6

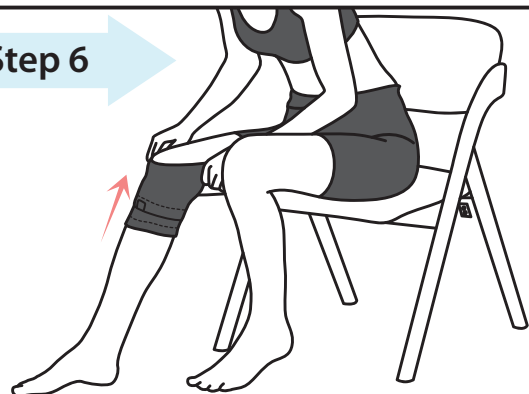

After folding the brace to half, then pull it up

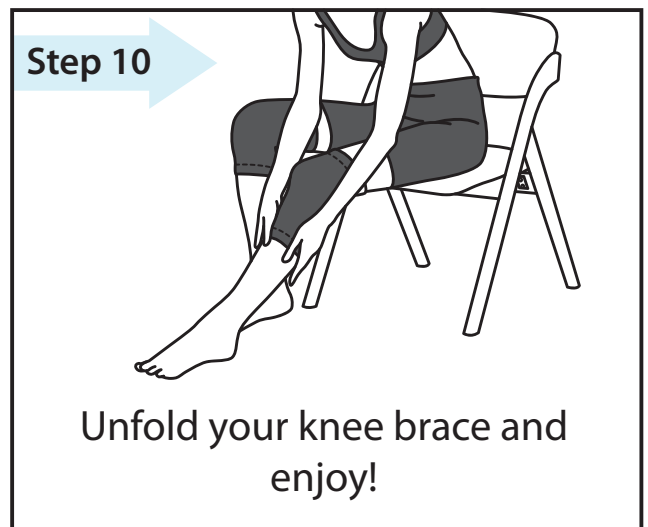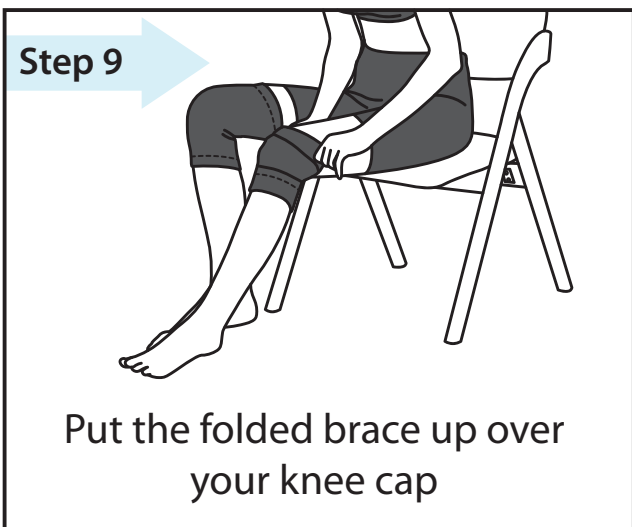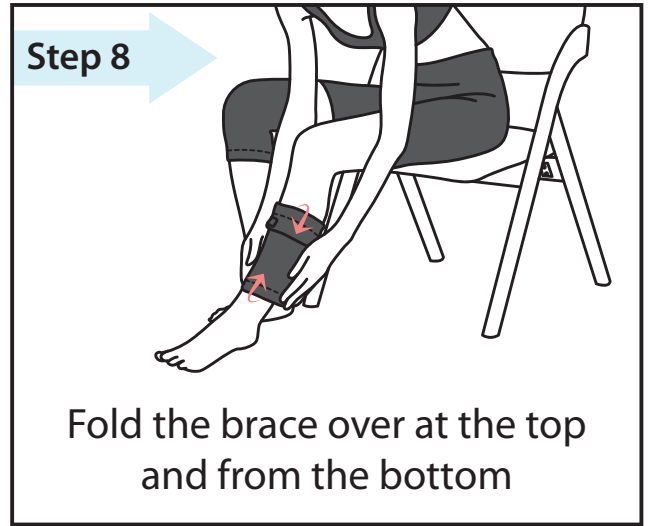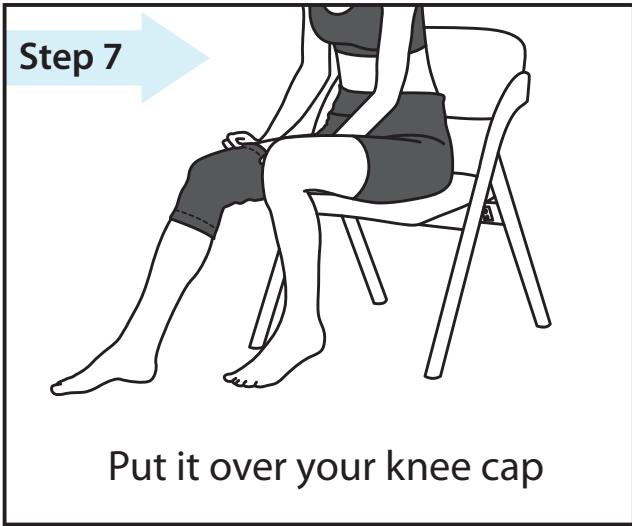

SIZE CHART

SIZE	INCHES	CM
XS	11.8" - 14.5"	30cm x 37cm
S	12.59" - 15.1"	32cm - 40cm
M	16.1" - 18.1"	41cm - 46cm
L	18.1" - 20.4"	46cm - 51cm
XL	20.4" - 22.8"	51cm - 59cm
XXL	23.3" - 26"	59cm - 68cm
XXXL	26" - 27.5"	68cm - 74cm